# RadiForce® MS231WT

Touch Panel Color LCD Monitor

Setup Guide Installationsanleitung Guide d'installation 设定指南

**Important :** Please read PRECAUTIONS, this Setup Guide and the User's Manual stored on the CD-ROM carefully to familiarize yourself with safe and effective usage. Please retain this guide for future reference.

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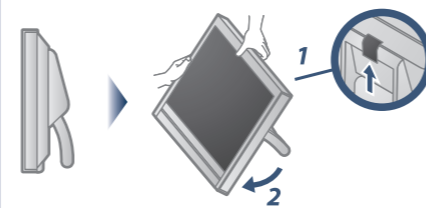
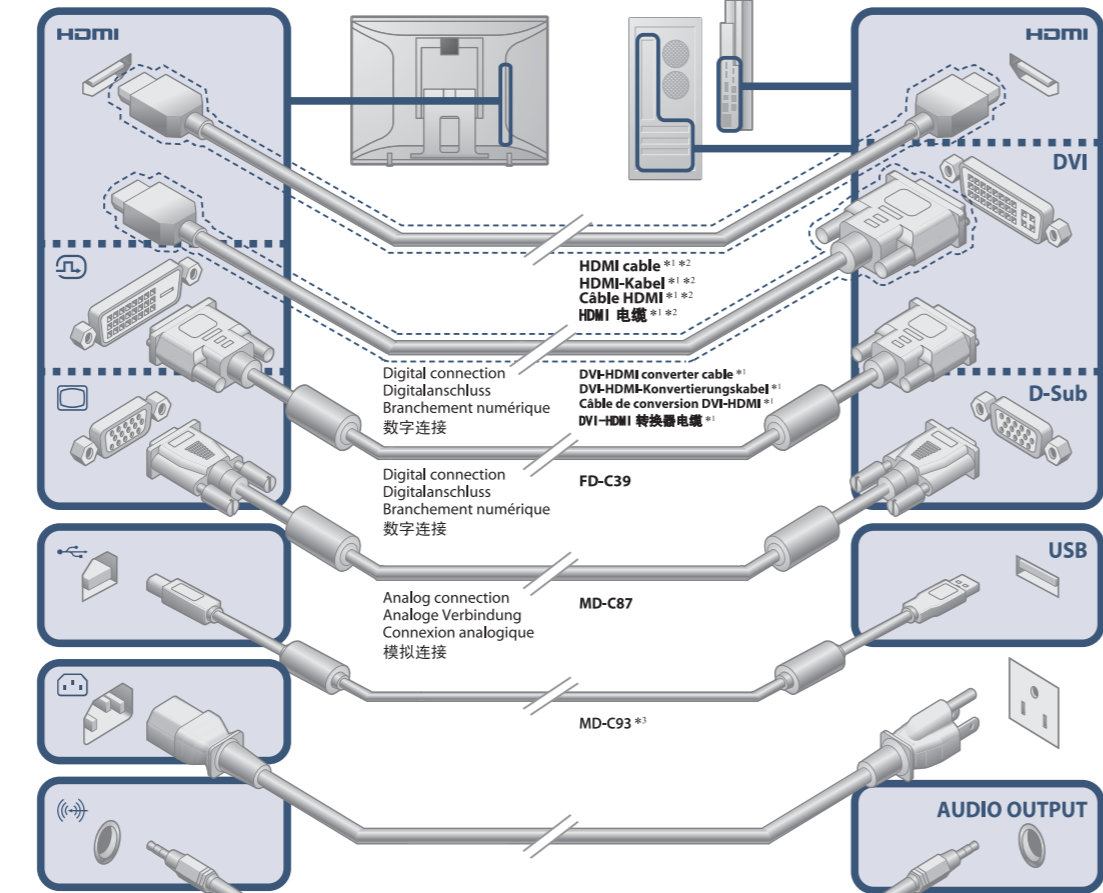
Touch pen  
Taststift  
Crayon tactile  
触摸笔

Cleaning cloth  
Reinigungstuch  
Chiffon de nettoyage  
清洁布

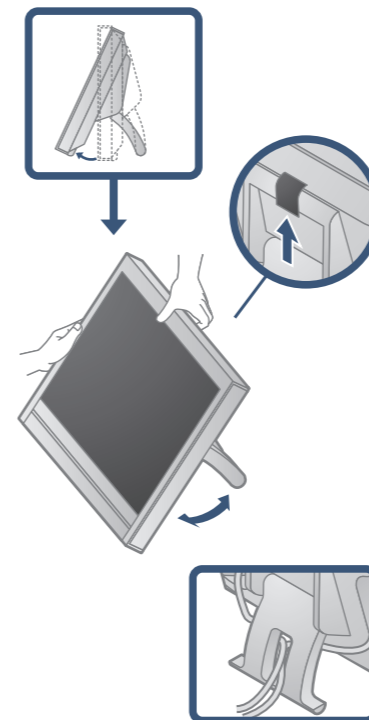
\*1 Commercially available product  
\*2 Use a product able to handle High Speed.  
\*3 Required when used as a touch panel monitor.

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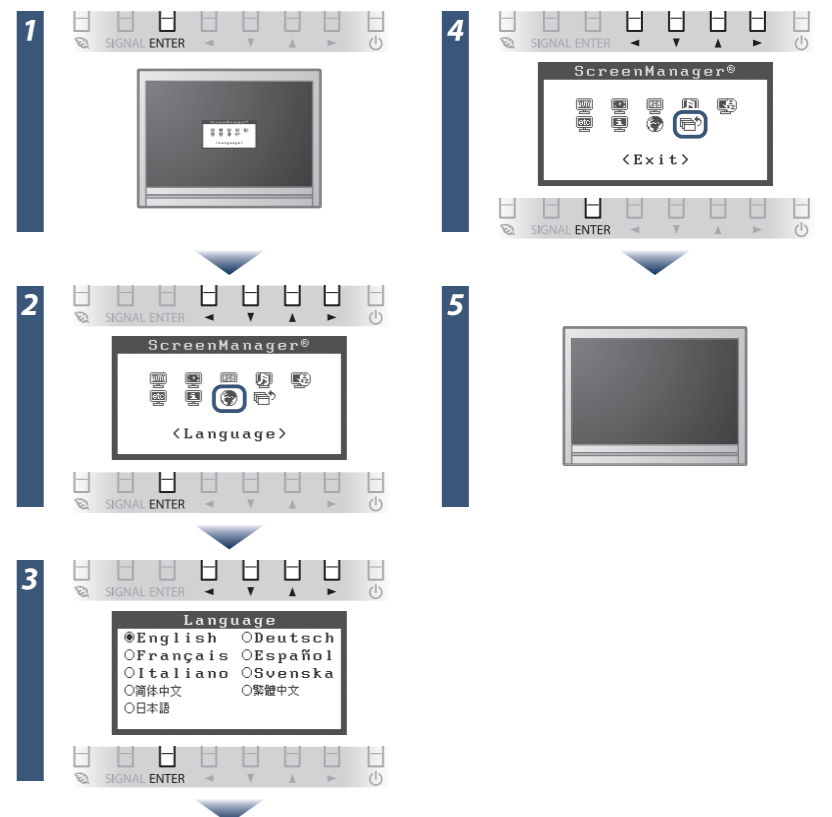
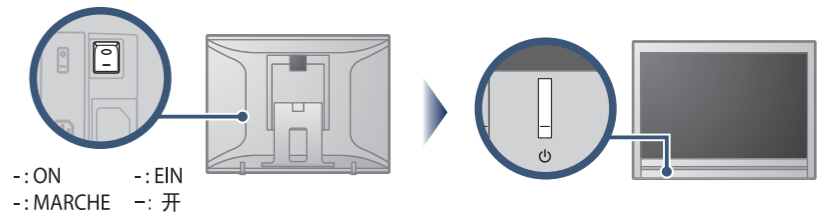
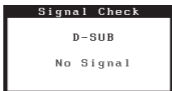
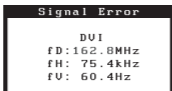
The touch panel driver must be installed when this product is used as a touch panel monitor. For details, refer to the installation guide for the touch panel driver (included on the CD-ROM).

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**No-Picture Problem** If a problem still remains after applying the suggested remedies, contact your local dealer.

Problems	Possible cause and remedy	Problems	Possible cause and remedy
Power indicator does not light.	<ul style="list-style-type: none"> <li>Check whether the power cord is connected properly.</li> <li>Turn off the main power, and then turn it on again a few minutes later.</li> <li>Turn the main power switch on.</li> <li>Press </li> </ul>	The message appears.	<p>This message appears when no signal is input. Example:</p>  <ul style="list-style-type: none"> <li>The message shown left may appear, because some external devices do not output the signal soon after power-on.</li> <li>Check whether the external device is turned on.</li> <li>Check whether the signal cable is connected properly.</li> <li>Switch the input signal with SIGNAL.</li> </ul>
Power indicator is lighting blue.	<ul style="list-style-type: none"> <li>Increase &lt;Brightness&gt;, &lt;Contrast&gt;, and/or &lt;Gain&gt; in the Adjustment menu.</li> </ul>		<p>The message shows that the input signal is out of the specified frequency range. (Such signal frequency is displayed in red.) Example:</p>  <ul style="list-style-type: none"> <li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor.</li> <li>Reboot the PC.</li> <li>Select an appropriate display mode using the graphics board's utility. Refer to the User's Manual of the graphics board for details.</li> </ul>
Power indicator is lighting orange.	<ul style="list-style-type: none"> <li>Switch the input signal with SIGNAL.</li> <li>Operate the mouse or keyboard.</li> <li>Check whether the external device is turned on.</li> </ul>		<p>fD : Dot Clock (Displayed only during digital signal input) fH : Horizontal Scan Frequency fV : Vertical Scan Frequency</p>

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