Installation Manual

RadiForce® MX191

Color LCD Monitor

Important

Please read this Installation Manual, and “Instructions for Use” (separate volume) carefully to familiarize yourself with safe and effective usage.

- Please refer to the “Instructions for Use” for basic information ranging from connection of the monitor to a PC to using the monitor.
- The latest Installation Manual is available for download from our web site:

<http://www.eizoglobal.com>

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

CONTENTS

CONTENTS	3		
Chapter 1 Basic Operations and Functions	4		
1-1. Basic Adjustment Menu Operations.....	4		
● Displaying the Adjustment Menu	4		
● Modifying the Setting	4		
● Displaying the Function Name	4		
● Storing the Setting/Exiting the Adjustment Menu	4		
1-2. Functions	5		
Chapter 2 Adjusting Screens	7		
2-1. Compatible Resolutions	7		
2-2. Setting the Resolution	7		
● Windows 10.....	7		
● Windows 8.1/Windows 8/Windows 7	7		
● Windows Vista.....	7		
● OS X Mountain Lion (10.8) and later.....	7		
● Mac OS X 10.7	8		
2-3. Adjusting Color	8		
● Selecting the Display Mode (CAL Switch Mode).....	8		
● Performing Advanced Adjustments	8		
● Adjustable Settings in Each Mode	9		
● Adjusting Brightness	9		
● Adjusting Contrast.....	10		
● Adjusting Temperature	10		
● Adjusting Gamma.....	11		
● Adjusting Gain.....	11		
● Adjusting Hue	12		
● Adjusting Saturation	12		
2-4. Adjusting Screens	13		
● Digital Signal Input	13		
● Analog Signal Input.....	13		
2-5. Adjusting Sharpness	16		
Chapter 3 Setting Monitor	17		
3-1. Setting Language.....	17		
3-2. Displaying or Hiding the EIZO Logo	17		
3-3. Locking Operation Buttons	18		
3-4. Resetting to Default.....	19		
● Resetting the Color Adjustments	19		
● Resetting All Adjustments to Defaults.....	19		
Chapter 4 Power Saving Functions	20		
4-1. Power Saving Mode	20		
● Digital Signal Input	20		
● Analog Signal Input.....	20		
4-2. Setting Power Indicator Display.....	20		
4-3. Setting Auto Brightness Adjustment of the Monitor.....	21		
4-4. Setting Auto Power-Off Function of the Monitor	21		
Chapter 5 Troubleshooting	22		
5-1. No Picture	22		
5-2. Imaging Problems (for both digital and analog).....	23		
5-3. Imaging Problems (for digital only)	23		
5-4. Imaging Problems (for analog only)	24		
5-5. Other Problems	24		
5-6. USB Problems	24		
Chapter 6 Reference	25		
6-1. Attaching Arm	25		
6-2. Connecting Multiple PCs	26		
● Switching among Input Signals.....	26		
● Setting Input Signal Switching Procedure	27		
6-3. Showing Monitor Information.....	27		
6-4. Main Default Settings	28		
6-5. Preset Timing	29		
LIMITED WARRANTY	30		

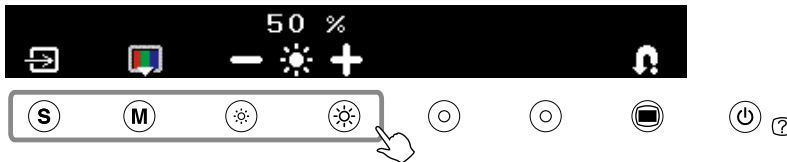
Chapter 1 Basic Operations and Functions

1-1. Basic Adjustment Menu Operations

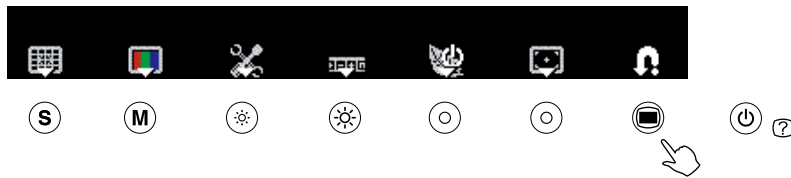
● Displaying the Adjustment Menu

Pressing the function button to change the setting or pressing  allows you to display the adjustment menu.

When pressing the input signal selection, mode or brightness control

When pressing the menu button

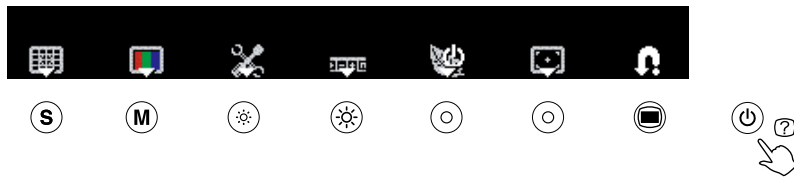
● Modifying the Setting

The operation methods differ depending on the icons.

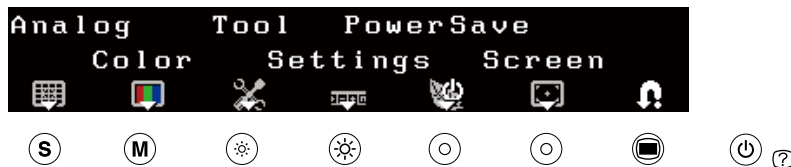
Display example	Operation method
	As for the icons displayed with the +/- marks, selecting + allows you to increase the setting value or selecting - to decrease it.
	Selecting the icon displayed with ▾ mark allows you to show the next menu.
	The Enable/Disable setting is toggled by selecting the icon. X is displayed when the function is invalid.

● Displaying the Function Name

1. Press  while the adjustment menu is displayed.

2. The function name is displayed.

● Storing the Setting/Exiting the Adjustment Menu

After completing the setting, selecting  allows you to store the setting. Selecting  repeatedly allows you to close the adjustment menu.

Note

- The adjustment menu automatically finishes if no input operations occur within 45 seconds.

1-2. Functions

The following table shows setting items of the adjustment menu.

Icon	Function	Overview
	-	Determines the setting. Selecting the icon repeatedly allows you to close the adjustment menu.
Analog		
	AutoSize	Flickering, screen position, and screen size are adjusted automatically. Returns to the pre-adjusted status. Determines the adjustment.
	AutoRange	Every color gradation (0 to 255) can be displayed by adjusting the signal output level. Returns to the pre-adjusted status. Determines the adjustment.
	Manual	Performs advanced adjustment when the automatic adjustment fails. Adjust the clock, phase and position, in this order.
	Clock	Eliminates vertical bars.
	Phase	Removes flickering or blurring.
	Position	Corrects the screen position Use to adjust the position.
Color		
	Color Menu	Independent setting and saving of color adjustment are available for each CAL Switch mode.
	Mode*	Selects the best display mode easily according to monitor's application. Each time you press the button, the mode is switched.
	Brightness	Adjusts the screen brightness. Adjustable range: 0 to 100 %
	Contrast	Adjusts the contrast. Adjustable range: 0 to 100 %
	Gain	Determines the mixing rate of red, green and blue, and adjust the color tone. Adjustable range: 0 to 100 % Allows you to determine the mixing rate of red. Allows you to determine the mixing rate of green. Allows you to determine the mixing rate of blue.
	Advanced	Adjusts each function of hue and saturation.
	Hue	The hue can be adjusted. If you set the hue to a high value, the image turns greenish. If low, the image turns purplish. Adjustable range: -100 to 100

Icon	Function	Overview
	Saturation	The color saturation can be adjusted. Adjustable range: -100 to 100
	Temperature	The color temperature can be adjusted. The gain preset values are set for each color temperature setting value. Adjustable range: Native, 6500K-15000K (specified by every 500K unit, including 9300K), User (at gain adjustment)
	Gamma	Gamma can be adjusted. Adjustable range: 1.8, 2.0, 2.2
Tools		
	Info1	Displays the input signal status (analog/digital), resolution, H/V frequency.
	Info2	Displays the model name, usage time and serial number.
	Input Signal	Sets to detect the input signal automatically. The Auto/Manual setting is toggled by selecting the icon.
	Reset	Resets the adjustment values to the factory settings. Attention • After resetting, you cannot undo the operation. All: Resets all adjustments to the default settings (factory settings). Color (CUS*): Only the adjustment values in the current CAL Switch mode will revert to the default settings (factory settings).
Menu Settings		
	Language	A language for the Adjustment menu and messages can be selected. The language setting is toggled by selecting the icon.
Power Save		
	Auto EcoView	The sensor on the front side of the monitor detects the environmental brightness to adjust the screen brightness automatically and comfortably by using the Auto EcoView function. The Enable/Disable setting is toggled by selecting the icon.
	Power LED	This function allows you to set the power indicator (blue) ON/OFF in the monitor ON condition. The On/Off setting is toggled by selecting the icon.

Icon	Function	Overview
	Off Timer	This function allows the monitor to automatically turn off after a specified time has passed. This function serves to protect the monitor from after-images caused when the same image is left displayed for a long time. Use this function when an image is displayed throughout the day. Adjustable range: Off, 1 to 23h
Screen		
	Sharpness	This function allows you to emphasize the outline of the image by emphasizing the color difference between the pixels composing the image and to improve a sense of quality and materiality. Contrarily, this function allows you to show the image smoothly by blurring the outline of the image.

*1 The name of the current CAL Switch mode is displayed.

Chapter 2 Adjusting Screens

2-1. Compatible Resolutions

For compatible resolutions of this product, refer to Instruction for Use "Compatible Resolutions".

2-2. Setting the Resolution

● Windows 10

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click "Display Settings".
3. On the "Customize your display" dialog box, click "Advanced display settings".
4. Select a monitor, then choose a resolution from the "Resolution" pull-down menu.
5. Click the "Apply" button.
6. When a confirmation dialog box is displayed, click "Keep changes".

● Windows 8.1/Windows 8/Windows 7

1. For Windows 8.1 / Windows 8, click the "Desktop" tile on the Start Screen to display the desktop.
2. Right-click the mouse anywhere on the desktop except for icons.
3. From the displayed menu, click "Screen Resolution".
4. On the "Screen Resolution" dialog box, select the monitor.
5. Click "Resolution" to select the desired resolution.
6. Click the "OK" button.
7. When a confirmation dialog box is displayed, click "Keep changes".

● Windows Vista

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click "Personalize".
3. On the "Personalization" window, click "Display Settings".
4. On the "Display Settings" dialog box, select the "Monitor" tab and select desired resolution in the "Resolution" field.
5. Click the "OK" button.
6. When a confirmation dialog box is displayed, click "Yes".

● OS X Mountain Lion (10.8) and later

1. Select "System Preferences" from the Apple menu.
2. When the "System Preferences" dialog box is displayed, click "Displays". (For Mac OS X Mountain Lion (10.8), click "Displays" for "Hardware".)
3. On the displayed dialog box, select the "Display" tab, then choose "Change" for "Resolutions".
4. The list of selectable resolutions is displayed. Select the desired resolution.
If the desired resolution is not displayed in the list, hold down the Option key and select "Change".
5. Your selection will be reflected immediately. When you are satisfied with the selected resolution, close the window.

● Mac OS X 10.7

1. Select “System Preferences” from the Apple menu.
2. When the “System Preferences” dialog box is displayed, click “Displays” for “Hardware”.
3. On the displayed dialog box, select the “Display” tab and select desired resolution in the “Resolutions” field.
4. Your selection will be reflected immediately. When you are satisfied with the selected resolution, close the window.

2-3. Adjusting Color

● Selecting the Display Mode (CAL Switch Mode)

A desired mode can be selected easily according to the monitor application.

Display Modes

Mode	Purpose
 (DICOM-CL)	DICOM-CL mode. Available for setting tailored to X-ray film color (clear base).
 (TXT)	Text mode. Suitable for displaying texts for word processing or spreadsheets.
 (CUS)	Custom mode. Available for the color settings according to your preference.
 (CAL)	CAL mode. Displays the screen adjusted by calibration software.

Procedure

1. Press .
The adjustment menu appears.
2. Press  again while the adjustment menu is displayed.
Each time you press the button, the mode is switched.
3. Select .
The mode selected is set.

←→ DICOM-CL → TXT → CUS → CAL →

Note

- You can also select the CAL Switch mode by pressing  and select  (Color).

● Performing Advanced Adjustments

Independent setting and saving of color adjustment are available for each CAL Switch mode.

Attention

- It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.
- Perform  (AutoRange) first when adjusting color for analog input signals. (See “To adjust color gradation automatically” (page 16).)
- The same image may be seen in different colors on multiple monitors due to their monitor-specific characteristics. Make fine color adjustment visually when matching colors on multiple monitors.

Note

- The values shown in “%” or “K” are available only as reference.

● Adjustable Settings in Each Mode

According to the CAL Switch mode selected, the adjustable function differs. (You cannot select any non-adjustable or non-settable function.)

√: Adjustable, - : Unadjustable

Icon	Function	CAL Switch mode			
		DICOM-CL	TXT	CUS	CAL
	Brightness	-	√	√	-
	Contrast	-	√	√	-
	Temperature	-	√	√	-
	Gamma	-	√	√	-
	Gain	-	-	√	-
	Saturation	-	-	√	-
	Hue	-	-	√	-

*1 If you adjust the calibration in this mode using the calibration kit exclusive for to this monitor (see RadiCS LE /“Accessories” in the “Instructions for Use”), only “Brightness” and “Color Reset” are adjustable.

● Adjusting Brightness

The screen brightness is adjusted by changing the brightness of the backlight (light source from the LCD back panel).

Setting Range

0 to 100 %

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Adjust the brightness with  or  of  (Brightness).
4. Select .
The adjustment is completed.

Note

- You can also adjust the brightness by pressing  or  while the adjustment menu is not displayed.

● Adjusting Contrast

The luminance of the screen is adjusted by varying the video signal level.

Setting Range

0 to 100 %

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Adjust the contrast with  or  of  (Contrast).
4. Select .
The adjustment is completed.

Note

- You can also adjust the contrast by pressing  while the adjustment menu is not displayed, and press  or  of  (Contrast).
 - At a contrast level of 50 %, every color gradation is displayed.
 - When adjusting the monitor, users are recommended to adjust the brightness, which may not downgrade the gradation characteristics, prior to adjusting the contrast. Perform the contrast adjustment in the following cases.
 - If you think the screen is dark even when the brightness level is set to 100 %
 - If you think the black level of the screen is bright (Reduce the brightness level and set the contrast level above 50 %)
-

● Adjusting Temperature

The color temperature can be adjusted.

The color temperature is normally used to express the hue of “white” and/or “black” by a numerical value. The value is expressed in degrees “K” (Kelvin).

The screen becomes reddish at a low color temperature, and bluish at a high color temperature, like the flame temperature. The gain preset values are set for each color temperature setting value.

Setting Range

Native, 6000K - 15000K (specified by every 500K unit, including 9300K), User (at gain adjustment)

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Select  (Color Menu).
4. Adjust the color temperature with  or  of  (Temperature).
5. Select .
The adjustment is completed.

Note

- “Gain” allows you to perform more advanced adjustments (refer to [“Adjusting Gain” \(page 11\)](#)).
 - If you set “Native”, the image is displayed in the preset color of the monitor (Gain: 100 % for each RGB).
 - When gain is changed, the color temperature is changed to “User”.
-

● Adjusting Gamma

Gamma can be adjusted. The brightness of the monitor varies depending on the input signal, however, the variation rate is not proportional to the input signal. To keep the balance between the input signal and the brightness of the monitor is called as “Gamma correction”.

Setting Range

1.8, 2.0, 2.2

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Select  (Color Menu).
4. Adjust the gamma value with  or  of  (Gamma).
5. Select .
The adjustment is completed.

● Adjusting Gain

The brightness of each color component red, green, and blue is called Gain. You can change the hue of “white” by adjusting gain.

Setting Range

0 to 100 %

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Select  (Color Menu).
4. Select  (Gain).
5. Adjust the respective value of  (Red)/  (Green)/  (Blue) with  or .
6. Select .
The adjustment is completed.

Note

- The value of gain changes with that of color temperature.
 - When gain is changed, the color temperature is changed to “User”.
-

● Adjusting Hue

The hue can be adjusted. If you set the hue to a high value, the image turns greenish. If low, the image turns purplish.

Setting Range

-100 to 100

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Select  (Color Menu).
4. Select  (Advanced).
5. Adjust the hue with  or  of  (Hue).
6. Select .
The adjustment is completed.

Attention

- After this adjustment, some color gradations may not be displayed.
-

● Adjusting Saturation

The color saturation can be adjusted.

Setting Range

-100 to 100

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Color).
3. Select  (Color Menu).
4. Select  (Advanced).
5. Adjust the saturation with  or  of  (Saturation).
6. Select .
The adjustment is completed.

Attention

- After this adjustment, some color gradations may not be displayed.
-

Note

- The minimum value (-100) changes the screen to monochrome.
-

2-4. Adjusting Screens

● Digital Signal Input

When digital signals are input, images are displayed correctly based on the preset data of the monitor, but if you perform more advanced adjustment, see ““2-3. Adjusting Color” (page 8)” and subsequent pages.

● Analog Signal Input

Attention

- It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.
- The Self Adjust (automatic screen adjustment) function does not work for the images under the resolution of 800 × 600 (SVGA).
- The self adjustment and auto adjustment functions work correctly when an image is fully displayed over the Windows or Macintosh display area. They do not work properly in the cases below:
 - when an image is displayed only on a part of the screen (command prompt window, for example)
 - when a black background (wallpaper, etc.) is in useAlso, these functions cannot work properly in some graphic boards.

The monitor screen adjustment is used to suppress flickering of the screen or adjust screen position and screen.

Note

- The self adjustment function works when all of the following conditions are satisfied:
 - When a signal is input into the monitor for the first time or when a resolution or vertical/horizontal scan frequency not used before is set.

If the screen is not displayed correctly even after performing the self adjustment operation, adjust the screen according to the procedures on the following pages to use the monitor comfortably.

How to Set

1. Perform the auto adjustment.

● To adjust flickering, screen position, and screen size automatically

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Analog).
3. Select  (AutoSize).
Flickering, screen position, and screen size are adjusted automatically.
4. After confirming the screen message, select  (OK).
If you select  (Undo), the monitor returns to the pre-adjusted status.

If the screen is not displayed correctly even after performing Auto Adjustment, perform the adjustments according to the procedures on the following pages. If the screen is displayed correctly, go to “5. Adjusting color gradation.” (page 16).

2. Prepare to use analog screen adjustment patterns.

Download the “Screen adjustment pattern files” from our site: <http://www.eizoglobal.com>

Note

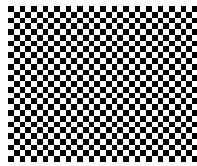
- For details and instructions on opening the “Screen adjustment pattern files”, refer to the Readme.txt file.
-

3. Perform the auto adjustment again with the analog screen adjustment pattern displayed.

● To adjust flickering, screen position, and screen size automatically

Procedure

1. Display Pattern 1 in full screen on the monitor using the “Screen adjustment pattern files”.

2. Press .
The adjustment menu appears.
3. Select  (Analog).
4. Select  (AutoSize).
Flickering, screen position, and screen size are adjusted automatically.
5. After confirming the screen message, select  (OK).
If you select  (Undo), the monitor returns to the pre-adjusted status.

If the screen is not displayed correctly even after performing Auto Adjustment, perform the adjustments according to the procedures on the following pages. If the screen is displayed correctly, go to “5. Adjusting color gradation.” (page 16).

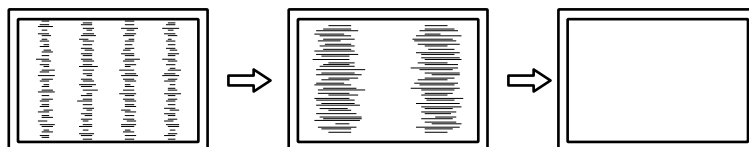
4. Choose from the adjustment menu to perform advanced adjustments.

Select  (Manual) to adjust the clock, phase and position, in this order.

● To eliminate vertical bars

Procedure

1. Adjust the clock with  or  of  (Clock) to eliminate vertical bars.

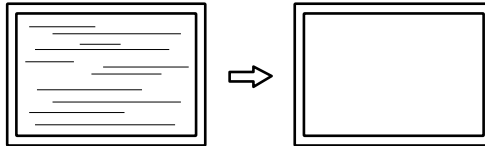
Note

- Press the control button slowly so as not to miss the adjustment point.
 - When blurring, flickering or bars appear on the screen after adjustment, proceed to “To remove flickering or blurring”.
-

● **To remove flickering or blurring**

Procedure

1. Adjust the phase with **−** or **+** of  (Phase) to remove flickering or blurring.

Attention

- Flickering or blurring may not be eliminated depending on your PC or graphics board.
-

● **To correct the screen position**

Note

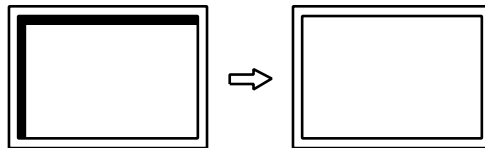
- Since the number of pixels and the pixel positions are fixed on the LCD monitor, only one position is provided to display images correctly. The position adjustment is made to shift an image to the correct position.
-

Procedure

1. Select  (Position).
2. Adjust the position with **←** (Left) / **→** (Right) / **↓** (Down) / **↑** (Up) to display the image properly in the display area of the monitor.

3. Select .

The adjustment is completed.

Note

- When vertical bars appear on the screen after adjustment, go back to “To eliminate vertical bars”. (Clock → Phase → Position)
-

4. Close Pattern 1.

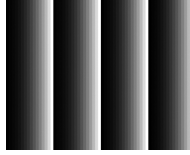
5. Adjusting color gradation.

● To adjust color gradation automatically

Every color gradation (0 to 255) can be displayed by adjusting the signal output level.

Procedure

1. Display Pattern 2 in full screen on the monitor using the “Screen adjustment pattern files”.

2. Press .
The adjustment menu appears.
3. Select  (Analog).
4. Select  (AutoRange).
Color gradation is adjusted automatically.
5. After confirming the screen message, select  (OK).
If you select  (Undo), the monitor returns to the pre-adjusted status.
6. Close Pattern 2.

2-5. Adjusting Sharpness

This function allows you to emphasize the outline of the image by emphasizing the color difference between the pixels composing the image and to improve a sense of quality and materiality. Contrarily, this function allows you to show the image smoothly by blurring the outline of the image.

Adjustable range

-3 to 3

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Screen).
3. Adjust the sharpness with  or  of  (Sharpness).
4. Select .
The adjustment is completed.

Chapter 3 Setting Monitor

3-1. Setting Language

A language for the Adjustment menu and messages can be selected.

Selectable languages

English/German/French/Spanish/Italian/Swedish/Simplified Chinese/Traditional Chinese/Japanese

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Settings).
3. The language setting is switched by selecting  (Language).

	English
	German
	French
	Spanish
	Italian
	Swedish
	Simplified Chinese
	Traditional Chinese
	Japanese

4. Select .
The language selected is set.

3-2. Displaying or Hiding the EIZO Logo

The EIZO logo can be displayed or hidden.

Procedure

1. Press  to turn off the monitor.
2. Hold down  and press  to turn on the monitor.
The EIZO logo display setting is toggled by performing the operation in step 2.

3-3. Locking Operation Buttons

The settings can be locked to prevent them being changed.

There are two types of locks: “Key Lock” and “Menu Lock”.

Key Lock: Locks buttons other than the Power button.

Menu Lock: Locks buttons other than the Power, Input, and Mode buttons.

Procedure

1. Press  to turn off the monitor.
2. Hold down  and press  to turn on the monitor.
The adjustment menu appears.
3. Each time you press , the mode is switched.

	Unlock
	Menu Lock
	Key Lock

4. Select .
The mode selected is set.

Attention

- The key lock function is also activated when calibration is performed using RadiCS/RadiCS LE.
-

3-4. Resetting to Default

There are two different Resets: Color Reset for only resetting the color adjustments to the default settings and All Reset for resetting all settings to the defaults.

Attention

- After resetting, you cannot undo the operation.

Note

- Regarding the default setting, see “6-4. Main Default Settings” (page 28).
-

● Resetting the Color Adjustments

Only the color adjustment value for the current CAL Switch mode can be reset to the default settings.

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Tool).
3. Select  (Reset).
If you will not perform the reset operation, select .
4. Select  (Color (CUS*¹)).
The color adjustment values revert to the default settings.
*1 The name of the current CAL Switch mode is displayed.

● Resetting All Adjustments to Defaults

All adjustments are reset to the default settings.

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Tool).
3. Select  (Reset).
If you will not perform the reset operation, select .
4. Select  (All).
All setting values revert to the default settings.

Chapter 4 Power Saving Functions

4-1. Power Saving Mode

Attention

- When not using the monitor, you can disconnect the power plug so that the power is completely cut.
- Five seconds before the monitor enters the power saving mode, it displays a preliminary message.

● Digital Signal Input

This monitor complies with the DVI DMPM standard.

Power Saving System

The monitor enters the power saving mode after five seconds in connection with the PC setting.

PC	Monitor	Power Indicator
On	Operation mode	Blue
Power saving mode	Power saving mode	Orange

Resumption Procedure

Operate the mouse or keyboard to resume the normal screen operations.

● Analog Signal Input

This monitor complies with the VESA DPM standard.

Power Saving System

PC	Monitor	Power Indicator
On	Operation mode	Blue
Power saving mode	STAND-BY SUSPENDED Off	Orange

Resumption Procedure

Operate the mouse or keyboard to resume the normal screen operations.

4-2. Setting Power Indicator Display

This function allows you to set the power indicator (blue) ON/OFF in the monitor ON condition.

Procedure

1. Press .
The adjustment menu appears.
2. Select  (PowerSave).
3. The On/Off setting is toggled by selecting  (Power LED).

	On
	Off

4. Select .
The mode selected is set.

4-3. Setting Auto Brightness Adjustment of the Monitor

The sensor on the front side of the monitor detects the environmental brightness to adjust the screen brightness automatically and comfortably by using the Auto EcoView function.

Attention

- This function is not available in the DICOM-CL or CAL modes.

Procedure

1. Press .
The adjustment menu appears.
2. Select  (PowerSave).
3. The Enable/Disable setting is toggled by selecting  (Auto EcoView).

	Enable
	Disable

4. Select .
The mode selected is set.

4-4. Setting Auto Power-Off Function of the Monitor

This function allows the monitor to automatically turn off after a specified time has passed. This function serves to protect the monitor from after-images caused when the same image is left displayed for a long time. Use this function when an image is displayed throughout the day.

Adjustable range

Off, 1 - 23h

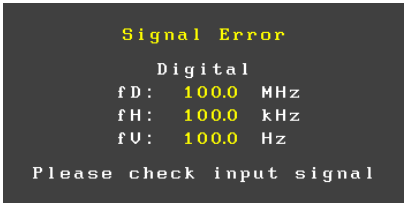
Procedure

1. Press .
The adjustment menu appears.
2. Select  (PowerSave).
3. Set a monitor OFF time with  or  of  (Off Timer).
Select "Off" to set Off timer setting to "Off".
4. Select .
The mode selected is set.
5. "The power will turn off soon." one minute before the setting time.
If you select  (Extend), the off timer function works 90 minutes later.

Chapter 5 Troubleshooting

If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

5-1. No Picture

Problem	Possible cause and remedy
<p>1. No picture</p> <ul style="list-style-type: none"> Power indicator does not light. Power indicator lights blue. Power indicator is lighting orange. 	<ul style="list-style-type: none"> Check whether the power cord is connected properly. Press . Increase "Brightness", "Contrast" and "Gain" in the adjustment menu (see "Performing Advanced Adjustments" (page 8)). Switch the input signal with . Move the mouse or press any key on the keyboard. Check whether the PC is turned on.
<p>2. The message below appears.</p> <ul style="list-style-type: none"> This message appears when no signal is input. Example:  <ul style="list-style-type: none"> The message shows that the input signal is out of the specified frequency range. (Such frequency is displayed in yellow.) Example:  <p>fD: Dot clock (Displayed only during digital signal input) fH: Horizontal scan frequency fV: Vertical scan frequency</p>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <ul style="list-style-type: none"> The message shown left may appear, because some PCs do not output the signal soon after power-on. Check whether the PC is turned on. Check whether the signal cable is connected properly. Switch the input signal with . Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the "Instructions for Use"). Reboot the PC. Select an appropriate setting using the graphics board's utility. Refer to the manual of the graphics board for details.

5-2. Imaging Problems (for both digital and analog)

Problem	Possible cause and remedy
1. The screen is too bright or too dark.	<ul style="list-style-type: none">• Use “Brightness” and “Contrast” in the adjustment menu to adjust it. (The LCD monitor backlight has a limited life span. When the screen becomes dark or begins to flicker, contact your local EIZO representative.)
2. Characters are blurred.	<ul style="list-style-type: none">• Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see “Compatible Resolutions” in the Instructions for Use).
3. Afterimages appear.	<ul style="list-style-type: none">• Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time.• Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.
4. Green/red/blue/white dots or defective dots remain on the screen.	<ul style="list-style-type: none">• This is due to LCD panel characteristics and is not a failure.
5. Interference patterns or pressure marks remain on the screen.	<ul style="list-style-type: none">• Leave the monitor with a white or black screen. The symptom may disappear.
6. The outlines of the letters are colored.	<ul style="list-style-type: none">• If the [Sharpness] setting is set to a high level, the characters displayed on the monitor may be colored.

5-3. Imaging Problems (for digital only)

Problem	Possible cause and remedy
1. Noise appears on the screen.	<ul style="list-style-type: none">• When entering the signals of HDCP system, the normal images may not be displayed immediately.
2. The auto adjust function does not work correctly.	<ul style="list-style-type: none">• This function does not work when a digital signal is input.

5-4. Imaging Problems (for analog only)

Problem	Possible cause and remedy
<p>1. Display position is incorrect.</p> 	<ul style="list-style-type: none"> Use "Position" in the Adjustment menu to correct the image position (see "To correct the screen position" (page 15)). If the problem persists, use the graphics board's utility if available to change the display position.
<p>2. Vertical bars appear on the screen or a part of the image is flickering.</p> 	<ul style="list-style-type: none"> Use "Clock" in the Adjustment menu to adjust it (see "To eliminate vertical bars" (page 14)).
<p>3. Whole screen is flickering or blurring.</p> 	<ul style="list-style-type: none"> Use "Phase" in the Adjustment menu to adjust it (see "To remove flickering or blurring" (page 15)).
<p>4. The auto adjust function does not work correctly.</p>	<ul style="list-style-type: none"> This function is intended for use on a Macintosh and on AT-compatible PC running Windows. It may not work properly in either of the following cases. It does not work properly when an image is displayed only on a part of the screen (command prompt window, for example) or when a black background (wallpaper, etc.) is in use. This function does not work correctly with some graphics boards.

5-5. Other Problems

Problem	Possible cause and remedy
<p>1. The adjustment menu does not appear.</p>	<ul style="list-style-type: none"> Check whether the operation lock function is active (see "3-3. Locking Operation Buttons" (page 18)).

5-6. USB Problems

Problem	Possible cause and remedy
<p>1. A monitor connected with a USB cable is not identified.</p>	<ul style="list-style-type: none"> Check whether the USB cable is connected properly. Reboot the PC. Check whether the PC and its OS are USB compliant. (For USB compliance of the respective devices, consult their manufacturers.) For Windows, check the PC's BIOS setting for USB. (Refer to the manual of the PC for details.)

Chapter 6 Reference

6-1. Attaching Arm

An optional arm (or an optional stand) can be attached by removing the stand section. For supported optional arms (or optional stands), refer to the EIZO website: <http://www.eizoglobal.com>

Attention

- When attaching an arm or stand, follow the instructions of their User's Manual.
- When using another manufacturer's arm or stand, confirm the following in advance and select one conforming to the VESA standard.
 - Clearance between the screw holes: 100 mm × 100 mm
 - Strong enough to support weight of the monitor unit (except the stand) and attachments such as cables.
- When using another manufacturer's arm or stand, use the following screws to secure it.
 - Screws supplied with this product (M4 × 12 mm)
- When using an arm or stand, attach it to meet the following tilt angles of the monitor.
 - Up 45 degrees, down 45 degrees (landscape, and portrait display rotated 90 degrees clockwise)
- Connect the cables after attaching an arm or stand.
- Do not move the removed stand up and down. Doing so may result in injury or device damage.
- The monitor and arm or stand are heavy. Dropping them may result in injury or equipment damage.
- When setting the monitor to the portrait style, turn the monitor screen 90° in clockwise direction.

Procedure

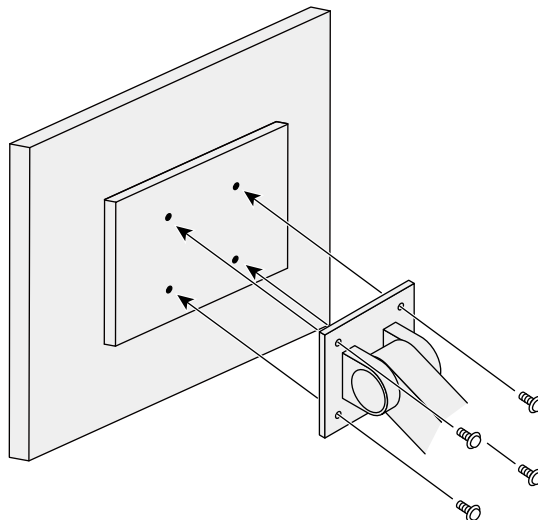
1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.

2. Remove the stand.

Prepare a screwdriver. Loosen the screws (four) securing the unit and the stand with the screwdriver.

3. Attach the arm or stand to the monitor.

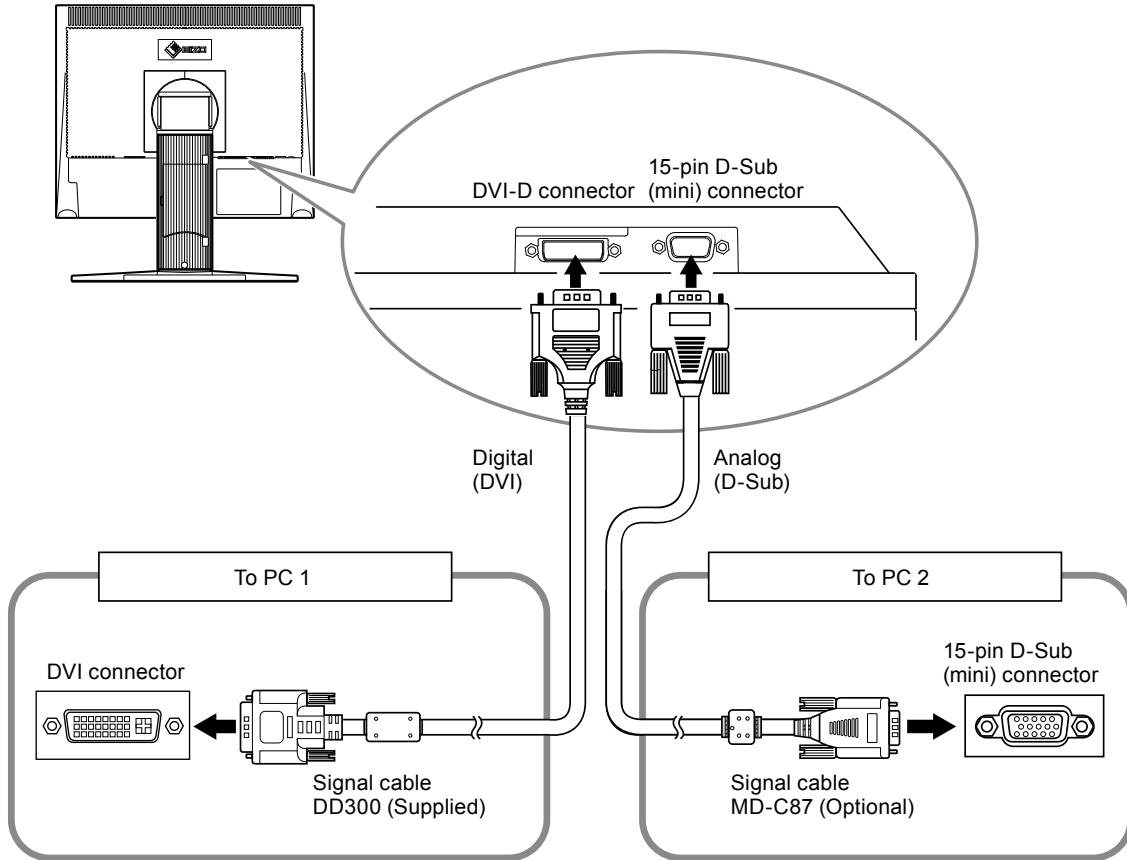
Use screws supplied with this product.

6-2. Connecting Multiple PCs

The product has multiple connections to PCs and allows you to switch to one of the connections for display.

Connection examples

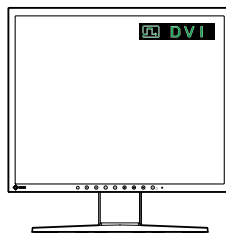
● Switching among Input Signals

Procedure

The input signal switches each time **(S)** is pressed.

When the signal is switched, the active input port name appears at the top right corner of the screen.

● Setting Input Signal Switching Procedure

Setting	Function
Auto	The monitor recognizes the connector through which PC signals are input. When a PC is turned off or enters the power saving mode, the monitor automatically displays another signal.
Manual	The monitor displays the signal from the selected connector despite the signal input. Select an active input signal with  .

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Tool).
3. The Auto/Manual setting is toggled by selecting  (Input Signal).

	Auto
	Manual

4. Select .
The mode selected is set.

Note

- If you select "Auto", the monitor enters the power saving mode after all the PCs have entered the power saving mode or have been switched off.

6-3. Showing Monitor Information

This function allows you to display the input signal status, current resolution and model name.

Info1: Displays the input signal status (analog/digital), resolution, H/V frequency.

Info2: Displays the model name, usage time and serial number.

Procedure

1. Press .
The adjustment menu appears.
2. Select  (Tool).
3. Select  (Info1) or  (Info2).
4. Select .
The menu is closed.

6-4. Main Default Settings

CAL Switch Mode: The default display setting is DICOM-CL.

Mode	Brightness	Temperature	Gamma value
DICOM-CL	170 cd/m ²	7500K	DICOM
Text	approx. 100 cd/m ²	7500K	2.2
Custom	approx. 300 cd/m ²	Native	2.2
CAL	170 cd/m ²	7500K	DICOM

Others

Auto EcoView		Disable
Off Timer		Disable
Input Signal		Auto
Languages		English
Sharpness		0

6-5. Preset Timing

The following table shows factory preset analog signal timing.

Attention

- Display position may be deviated depending on the PC connected, which may require screen adjustment using Adjustment menu.
- If a signal other than those listed in the table is input, adjust the screen using the Adjustment menu. However, screen display may still be incorrect even after the adjustment.
- When interlace signals are used, the screen cannot be displayed correctly even after screen adjustment using the Adjustment menu.

Resolution	Frequency	
	Horizontal: kHz	Vertical: Hz
640 × 400	24.83	56.42
640 × 400	31.48	70.10
640 × 480	31.47	59.94
640 × 480	37.86	72.81
640 × 480	37.50	75.00
720 × 400	31.47	70.09
800 × 600	35.16	56.25
800 × 600	37.88	60.32
800 × 600	48.08	72.19
800 × 600	46.88	75.00
1024 × 768	48.36	60.00
1024 × 768	56.48	70.07
1024 × 768	60.02	75.03
1152 × 864	67.50	75.00
1280 × 960	60.00	60.00
1280 × 1024	63.98	60.02
1280 × 1024	79.98	75.03

LIMITED WARRANTY

LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual attached to the Product (hereinafter referred to as the “**User’s Manual**”).

The period of this Warranty is limited to (i) five (5) years from the date of purchase of the Product; or, limited to the expiration of (ii) 30,000 hours of its use of the Product (hereinafter referred to as the “**Warranty Period**”); provided, however, that the Warranty Period for the LCD panel of the Product is limited to three (3) years from the date of purchase of the Product. EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts (excluding design parts) of the Product upon expiration of seven (7) years after the production of such parts is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards. If the unit cannot be repaired due to its condition or the stockout of a relevant part, EIZO and Distributors may offer the replacement by a product with equivalent performance instead of repairing it.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor;
- (d) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in brightness, changes in brightness uniformity, changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.);
- (e) Any defect of the Product caused by external equipment;
- (f) Any defect of the Product on which the original serial number has been altered or removed;
- (g) Any normal deterioration of the product, particularly that of consumables, accessories, and/or attachments (e.g. buttons, rotating parts, cables, User’s Manual, etc.); and
- (h) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

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